


Easy Steps to Apply Brijjit

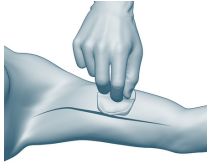

STEP 1


Prep the Skin: Once you are ready to apply your Brijjits, prepare the skin on both sides of the incision with a sterile alcohol pad or swab to remove oils, dead skin cells, and optimize adhesion. Using a rubbing motion is the most effective.

This step is IMPORTANT - Do Not Skip




STEP 2

Load the Device: Holding the applicator gently (do not squeeze), push the applicator pins into the open windows on the top of the Brijjit device until it snaps into place; you will hear an audible click that confirms proper loading.

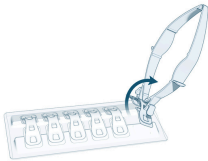


STEP 3

Roll Off Tray: While still holding the applicator gently, roll the loaded Brijjit device off the tray as shown (do not squeeze).

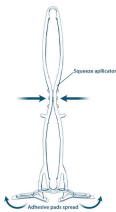

STEP 4

Activate & Place: Now while squeezing the applicator to activate the Brijjit device in preparation for placement, align the device over the incision and press the device down firmly onto the skin, setting the adhesive under the struts.

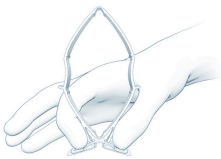

STEP 5

Release and Secure the Adhesive: Press the release tabs at the base of the applicator to detach the Brijjit device, now press down on the flat ends.



STEP 6

Repeat: To apply additional devices, repeat the above process, spacing the Brijjits on the patient like they are spaced on the tray.

Please Handle with Care.
Brijjit should be kept in a cool dry place.

Brijjit should be used at the direction of a licensed health care professional. Not be used on open wounds. For any questions, please contact your health care provider.